
- **Gebrauchsanweisung**
- **User's Manual**
- **Istruzioni per l'uso**
- **Mode d'emploi**
- **Instrucciones de empleo**
- **Manual de operação**
- **Руководство по применению**

Bedienung KaWe EUROLIGHT® D30 2,5 V und 3,5 V Dermatoskop:
Operation KaWe EUROLIGHT® D30 2,5 V and 3,5 V dermatoscopes:
Utilisation dermatoscopes KaWe EUROLIGHT® D30 2,5 V et 3,5 V:
Uso specifico dermatoscopi KaWe EUROLIGHT® D30 2,5 V e 3,5 V:
Modo de empleo dermatoscopios KaWe EUROLIGHT® D30 2,5 V y 3,5 V:
Operação dos dermatoscópios KaWe EUROLIGHT® D30 2,5 V e 3,5 V:
Обслуживание KaWe EUROLIGHT® дерматоскоп D30 /2,5 В и 3,5В: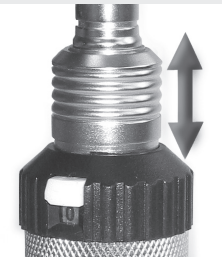

KIRCHNER & WILHELM GmbH + Co. KG
Eberhardstr. 56 • 71679 Asperg • Germany

Zentrale / Switchboard

Fon: +49-7141-68188-0 • Fax +49-7141-68188-11
eMail: info@kawemed.de • Internet: www.kawemed.de

QM-1-101C

2011/03

Kontaktsscheibe mit Skalierung für Dermatoskop Ø 25 mm
Contact disk with scale setting for dermatoscopes Ø 25 mm
Rondelle de contact avec cadran pour dermatoscopes Ø 25 mm
Vetretto di contatto con scala graduata per dermatoscopi Ø 25 mm
Arandela de contacto con escala para dermatoscópios Ø 25 mm
Disco de contacto com escala para dermatoscópios Ø 25 mm
Контактное стекло с маркировкой шкалы Ø 25 мм

Fokuseinstellung
Focus adjustment
Plage de focalisation
Messa a fuoco
Marge de la focalización
Ajuste do foco
Регулирование

Ladebatterie (Akku)
Rechargeable battery
Pile rechargeable (accu)
Batteria ricaricabile (accumulatore)
Pilha recarregável (acumulador)
Аккумулятор

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**User's Manual
Dermatoscope**

Dear customers, thank you for choosing a KaWe product. Our products are known for their high quality and long life. This KaWe product meets EC standards 93/42/EEC (standards for medical products).

Please read these instructions thoroughly and carefully before attempting to use this product and pay attention to the given instructions!

Use: The dermatoscopes are only to be used by authorized personnel.

Purpose: The KaWe dermatoscope is destined for the optical examination of the body's surface. It serves as a tool for the approx. 10-time magnification of the skin's surface and illuminates the examined dermatomatic area with a halogen bulb that is comparable with reflected light microscopic optics (epiluminescence microscopic optics). The dermatoscopes serve as a tool for the examination of pigmentary skin modifications, especially for the early diagnosis of malignant tumours. By moistening the skin with the special dermatoscope gel (KaWe Derma-Gel) cutaneous reflexes are avoided and the disturbing skin surfaces will be made invisible so that only the pure skin discoloration is recognizable. The unit is exclusively destined for the examination of the body's surfaces.

Dermatoscopes
- KaWe EUROLIGHT® D30 Dermatoscope 2,5 V
- KaWe PICCOLIGHT® D Dermatoscope 2,5 V
- KaWe EUROLIGHT® D30 Dermatoscope 3,5 V / 230 V (EU)
- KaWe EUROLIGHT® D30 Dermatoscope 3,5 V / 110 V (US)

Unsuited use/ contraindication of the products: Any other use is not in accordance with the destination of the unit. The manufacturer is not liable for any resulting damages. The user alone bears the risk. Only use the enclosed handle for electric power supply to the unit.

Operation KaWe EUROLIGHT® D30 2.5 V and 3.5 V dermatoscopes: EA satisfactory operation is guaranteed with the use of KaWe standard handles 2.5 V with KaWe accumulators 2.5 V or 3.5 V and/or with 2 batteries "C" Baby 1.5 V. Slip the instrument head on and lock it into place. Turn on the sliding switch by pushing and moving the white plastic button to the left. The light intensity (0=off 1=maximum brightness) can be easily regulated with the rheostat. Turn off the sliding switch by moving it back to the right to its starting point (switch motion). Change the batteries/accumulators by unscrewing the bottom locking cap. Handles with accumulator operation can be charged with the charging station KaWe MedCharge® 4000 by removing the plastic cap at the handle's bottom and plugging it into the charging slot of the KaWe MedCharge® 4000.

When removing the two twisted handle parts of the KaWe EUROLIGHT® D30 3.5 V (EU- and US version), the lower accumulator handle can be plugged into the power supply socket (ATTENTION: 120V/3.5V US version or 240V/3.5V EU version!!!). Please only use 3.5 V KaWe accumulators. The instrument head can be removed by pushing the sleeve down.

CAUTION:
Please discharge the NiMH accumulators completely before recharging them.

Operation KaWe PICCOLIGHT® D dermatoscope: A satisfactory operation will be guaranteed with the use of KaWe PICCOLIGHT® handles with 2 batteries ("AA"-Mignon 1.5V) Activate the on/off sliding switch. Turn the unit on using the sliding switch by moving it towards the dermatoscope head. Turn off the unit by sliding the switch in the opposite direction and back to its starting point. Change the battery and the lamp by removing the KaWe PICCOLIGHT®-head.

Replacing the lamp in the KaWe PICCOLIGHT® Dermatoscope: Satisfactory operation of the unit can only be guaranteed with the use of original KaWe incandescent lamps that ensure a good light quality. Remove the dermatoscope head from the handle and extract the lamp by pulling it out. If necessary, clean the glass bulb of the new lamp with alcohol. The glass bulb must be properly clean and free of finger prints (free of grease). The lamp must be pushed in carefully in the provided slot until it touches the lateral pin. Next, screw the instrument head onto the handle.

Replacing the lamp in the KaWe EUROLIGHT® Dermatoscope: The satisfactory operation of the dermatoscope can only be guaranteed with the use of original KaWe incandescent lamps that ensure a good light quality. Remove the dermatoscope head from the handle and remove the lamp by pulling it out. Then, proceed in the exact same manner as described above for the KaWe PICCOLIGHT® lamp exchange.

Use: Moisten the dermatomatic area that is to be examined with the special dermatoscope gel KaWe Derma-Gel. Switch on the illumination and place the dermatoscope so that the contact disk rests lightly on the dermatomatic area. You will obtain a uniform intensive optical stimulation of the skin with a large examination view (Ø 25 mm). The area that is to be examined should be in about the middle of the contact disk. The eye should be as close to the ocular as possible. Turn the focussing ring with your free hand until a perfectly focused view is reached. Try to look through the ocular with a relaxed eye. The contact disk with the scale setting measures pigmented skin modifications. In the case of elevated skin or skin that is sensitive to the touch (injured) the dermatoscope can be used without the contact disk. In order to ensure a precise view, the dermatoscope must be placed within a maximum distance of 5 mm of the skin.

Technical data:
Magnifying glass with approx. 10-fold magnification
Contact disk diameter: 25 mm
Focussing area/focus adjustment: -6 to +3.5 dpt
Light intensive halogen bulb 2.5 V and/or 3. V (KaWe EUROLIGHT®)
Light intensive vacuum bulb 2.5 V (KaWe PICCOLIGHT® D)

Further information, maintenance, storage:
When used and stored properly, the dermatoscope will serve you many years.

Guarantee: When used under normal circumstances and with attention to these instructions, we guarantee this product for two years after purchase date (except lamps and batteries/accumulators). Should you need further information or your instrument needs to be repaired, please contact your dealer!

Common accessory parts: Further information about dermatoscopes can be attained in the KaWe price list or at our Internet-Shop at: www.kawemed.de.

Manufacturer: **KaWe**

Warning: Never bring the dermatoscope in contact with liquid and make sure that no liquid penetrates the housing! Since the dermatoscope is not designed for operations, simple cleaning and/or disinfection with a surface disinfectant with an alcoholic base is sufficient. Do not use any scrubbing cleansing agents! The dermatoscope cannot be sterilized.

Limitation of reuse preparation: No special requirements. It is recommended that the instruments be repaired immediately after their use.

Instructions
Workstation: Remove dirt accumulation on surfaces with a disposable cloth.
Storage and transport: No special requirements.
Cleaning preparation: No special requirements. The instrument need not be disassembled.
Manual cleaning: In order to avoid dirt accumulation and dust, always store the dermatoscope in its packaging. The outside of the unit can be cleaned with moist, soft and lint-free fabric. For disinfection, a cloth moistened with alcohol can be used.
Automatic cleaning: The contact disk must be disinfected in alcohol or be boiled after each use.
Disinfection: No special requirements. The disinfectant can be used in accordance with the instructions given on the label.
Maintenance: Defective parts must be sorted out and/or replaced.
Check and performance test: The smooth running of all movable parts like the focus adjustment must be ensured. Check all nonrepairable items like lamps and batteries/accumulators.
Packaging: Individual packaging in bags or cardboard boxes.
Sterilization: Not a surgical instrument, no sterilization necessary or possible.
Storage: No special requirements.
Additional information: No special requirements.
Contacting the manufacturer:
Address or phone number of the dealer or dial +49-7141-68188-0.

Explanation of the symbols:

	Manufacturer		Separate collection of electrical and electronic devices
	Production date		Battery disposal
	See user's manual		Fulfillment of the relevant EU Rules
	Warning		

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